

Software Installation Notes

The Drivers and Utilities Disk includes the latest USB device drivers and interface library needed to operate your ScanX Digital Imaging System with In-Line Erase. It also includes diagnostic utilities and a demonstration program to help troubleshoot any problems you may encounter.

System Requirements

The Computer System (laptop or computer, monitor, etc.) and any related peripheral or equipment must comply with the requirements for accessory Equipment as specified in the appropriate IEC 60601-1 and/or IEC 60601-1-1 harmonized national standard.

The minimum computer system requirements necessary to operate the ScanX are listed below:

- Operating System: Microsoft Windows XP Professional for an Intel 32-bit processor with Service Pack 2, or later;
Microsoft Windows XP Professional 64-bit Edition for an Intel 64-bit extended (x64) processor with Service Pack 1, or later;
Microsoft Vista Business, Enterprise, or Ultimate for an Intel 32-bit processor with Service Pack 1, or later; or
Microsoft Vista Business, Enterprise, or Ultimate for an Intel 64-bit extended (x64) processor with Service Pack 1, or later.

USB Port: Must be USB 1.1 or later

Image Management Software: Authorized third-party Digital Imaging Software is required.


Installing the Software

NOTES: You must have Administrative privileges in order to install the device drivers, libraries, and other software.

Use the installation program to install the device drivers, the library files, and the ScanX utilities.

The software installation program can be started from the menu that automatically starts when the Drivers and Utilities disk is inserted into your CD-ROM player. You can start the menu manually by double-clicking on the AutoRun program in the root directory of the disk, typically "D:\AutoRun.exe". From the menu, click on "Install the Drivers and Utilities" to start the software installation program.

If you choose to install the device drivers (the default), Windows may report ask you to confirm that you want to install the device software. Press the 'Yes' or 'Install' button to complete the installation.

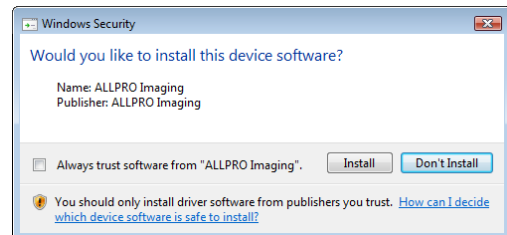

Updating the Library Files

If you install Authorized third-party Digital Imaging Software after installing the drivers and utilities, that software may replace the library files with older ones that are not fully compatible with your scanner. To restore the library files to the latest settings, run the Library Update program. This can be located by pressing the 'Start' button and looking for 'ScanX Utilities' under 'Programs', and then selecting 'Update Library'. An Update directory is also provided on the disk.